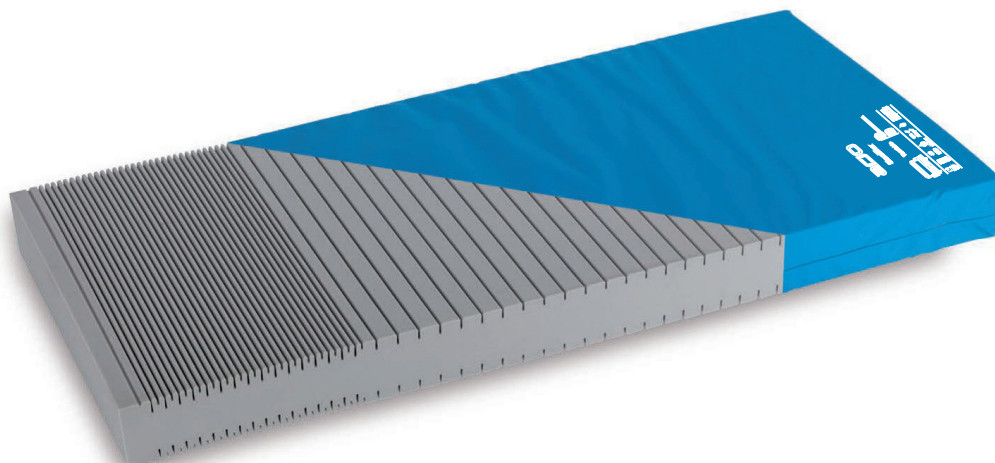
INSTRUCTIONS FOR USE

Simulflex

2-way turn mattress overlay

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**WARNING**

To avoid injury, always read this Instructions For Use and accompanied documents before using the product.

Mandatory to read the Instructions For Use.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

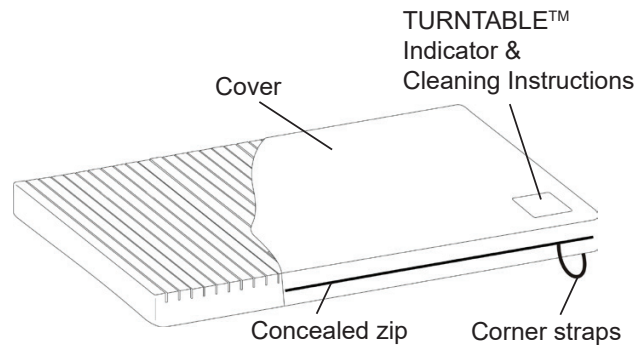
© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

Product Descriptions

Mattress replacement system

The SIMULFLEX™ overlay design comprises of a 2-way turn, contoured polyurethane foam with a 2-way stretch, vapour permeable, water resistant cover.

Product Descriptions

Mattress replacement system

The SIMULFLEX™ overlay design comprises of a 2-way turn, contoured polyurethane foam with a 2-way stretch, vapour permeable, water resistant cover.

Clinical applications

Indications for Use

SIMULFLEX overlay is suitable for comfort only.

WARNING

The cover of this product is vapour permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

Instructions for Use

1. Remove all packaging.

WARNING

The mattress overlay must be placed with the contoured surface facing up.

2. Lay the SIMULFLEX mattress overlay on top of a normal mattress ensuring that there are no sharp edges which may rip the cover.
3. Use the four corner straps to safely secure the overlay to the underlying mattress.
4. Ensure that the correct month is showing on the TURNTABLE™ indicator at the foot end of the mattress. If the correct month is not showing, rotate the mattress so that the correct month

is at the foot end. Months are indicated as follows:

1/12 = January

2/12 = February etc

5. Use the TURNTABLE™ indicator as a guide for the monthly rotation and turning of the mattress. This helps to even out wear of the mattress and to extend the life of the product.

CAUTION

Do not use the mattress overlay without a cover. It provides a protective barrier against biological contamination of the foam.

Decontamination

To Clean the Cover

Wipe all exposed surfaces with a cloth which has been moistened in simple detergent and water. Ensure to remove any organic debris that may be adherent to the surfaces. Dry thoroughly.

To Disinfect the Cover

For disinfection we recommend a chlorine-releasing agent such as NaOCl or NaDCC diluted to 1,000 ppm (range may vary between 250 & 10,000 ppm depending on contamination status and local policy).

Wipe with a cloth moistened with clean water and dry thoroughly.

Alternatively, use a cloth moistened with alcohol (70%) all allow to air dry.

Routine checks

The foam core and cover should be inspected regularly for any signs of damage, soiling or staining.

Technical Data

Typical bed size			Acute Single Size			Community Standard Size	Community Double Size
Suitable for use on Arjo Bedframe	20050C Child Anniversary™	Contoura™ 420 and 430	Contoura 350, 360, 460, 480, 560 and Minuet® 2	Minuet 1	Contoura 880 and 980		
Size L x W x D (cm)	170 x 86 x 8.5	182 x 78 x 8.5	198 x 86 x 8.5	200 x 90 x 8.5	202 x 88 x 8.5	190 x 92 x 8.5	190 x 138 x 8.5
SIMULFLEX Overlay Product Code	SO170/086	SO182/078	SO198/086	SO200/090	SO202/088	SO190/092	SO190/138
Material	Polyurethane foam						
Cover material	Polyurethane coated polyester fabric with enhanced durability						

SIMULFLEX overlays are available in other custom made sizes, e.g. SOYYYY/ZZZ, where YYY specifies the length (between 140 and 216cm) and ZZZ specifies the width (between 66 and 140cm). All sizes must be specified in increments of 2cm.

When combining beds and mattresses outside the table given above, it is the customer's responsibility to ensure compatibility of mattress and bed frame. Please refer to your bed manufacturers recommended mattress sizes. For further details contact Arjo.

Fire retardancy:

System meets the requirements of BS EN 597-1: 1995, BS EN 597-2: 1995 and BS 6807: 2006, 5.

FIRE WARNING

Cigarettes, lighters, naked flames and exposed heating elements e.g. in electric fires (and other electrical appliances) are sources of fire hazard and could ignite bed linen, blankets and clothing. It is essential that these fire sources are kept away from the mattress.

NB: It is highly dangerous to smoke in bed.

Disposal

- For ultimate disposal of SIMULFLEX products, we recommend the following advice be followed. However, regardless of this, please ensure that local regulations and, if applicable, healthcare facility standard operating procedures are followed.
- Foam cores suspected of contamination should be disposed of as clinical waste.
- Foam cores not suspected of contamination may be disposed of through landfill waste

Note: Follow all local regulations and, if applicable, healthcare facility standard operating procedures.

Warranty and Service

Standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance, and any questions regarding this, or any other product, please contact your local distributor.

Symbols

Recommended wash temperature:
15 min at 60°C
Maximum wash temperature:
15 min at 95°C

Do not iron

Do not use phenol-based cleaning
solutions

Hospital name

Tumble dry at 60°C
Maximum drying temperature 80°C

Wipe all surfaces with cleaning solution,
then wipe with a cloth moistened
with water and dry thoroughly.

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Pravidelná dezinfekcia: 1 000 ppm.

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Intentionally left blank

AUSTRALIA
Arjo Australia
Building B, Level 3
11 Talavera Road
Macquarie Park, NSW, 2113, Australia
Phone: 1800 072 040

BELGIQUE / BELGIË
Arjo Belgium
Evenbroekveld 16
9420 Erpe-Mere
Phone: +32 (0) 53 60 73 80
Fax: +32 (0) 53 60 73 81
E-mail: info.belgium@arjo.com

BRASIL
Arjo Brasil Equipamentos Médicos Ltda
Rua Marina Ciufuli Zanfelice, 329 PB02
Galpão - Lapa
São Paulo – SP – Brasil
CEP: 05040-000
Phone: 55-11-3588-5088
E-mail: vendas.latam@arjo.com
E-mail: servicios.latam@arjo.com

CANADA
Arjo Canada Inc.
90 Matheson Boulevard West
Suite 300
CA-MISSISSAUGA, ON, L5R 3R3
Tel/Tél: +1 905 238 7880
Free: +1 800 665 4831 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 905 238 7881
E-mail: info.canada@arjo.com

ČESKÁ REPUBLIKA
Arjo Czech Republic s.r.o.
Na Strži 1702/65
140 00 Praha
Czech Republic
Phone No: +420225092307
e-mail: info.cz@arjo.com

DANMARK
Arjo A/S
Vassingerødvej 52
DK-3540 LYNGE
Tel: +45 49 13 84 86
Fax: +45 49 13 84 87
E-mail:
dk_kundeservice@arjo.com

DEUTSCHLAND
Arjo GmbH
Peter-Sander-Strasse 10
DE-55252 MAINZ-KASTEL
Tel: +49 (0) 6134 186 0
Fax: +49 (0) 6134 186 160
E-mail: info-de@arjo.com

ESPAÑA
ARJO IBERIA S.L.
Poligono Can Salvatella
c/ Cabanyes 1-7
08210 Barberà del Valles
Barcelona - Spain
Telefono 1: +34 900 921 850
Telefono 2: +34 931 315 999

FRANCE
Arjo SAS
2 Avenue Alcide de Gasperi
CS 70133
FR-59436 RONCQ CEDEX
Tél: +33 (0) 3 20 28 13 13
Fax: +33 (0) 3 20 28 13 14
E-mail: info.france@arjo.com

HONG KONG
Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung,
N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA
Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST
Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

NEDERLAND
Arjo Nederland BV
Biezenwei 21
4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0) 344 64 08 00
Fax: +31 (0) 344 64 08 85
E-mail: info.nl@arjo.com

NEW ZEALAND
Arjo Ltd
34 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 5344
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@Arjo.com

NORGE
Arjo Norway AS
Olaf Helsets vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjo.com

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At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

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Arjo Inc.