

Maquet Lyra Mobile Operating Table

A versatile table for all-around use







Versatility for procedures

Reliability and user-friendly operation

Get the stability and functionality you need across surgical disciplines.


Around the world, the cost of healthcare is skyrocketing. As pressure on your budget is rising, it's increasingly important to invest in reliable, multifunctional infrastructure. That's why Getinge developed the Maquet Lyra Mobile OR Table. Just one table can meet the needs of most surgical disciplines, and is carbon-fiber accessory compatible for excellent intra-operative diagnostics.

Getinge is built on a genuine compassion for people's health, safety, and wellbeing. Founded in 1904 with roots dating back to 1838, Getinge has grown organically and through acquisitions to become a global market leader. Our portfolio offers solutions and support throughout the clinical pathway, and features well-known and dependable product brands, including Maquet OR Tables, that have been trusted for 180 years.



Cost-effective compatibility

Giving surgeons the options they need

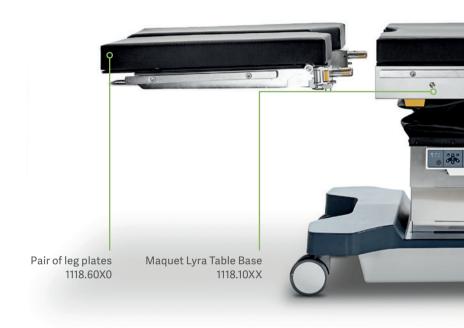



Maximum overall load

The Maquet Lyra Mobile OR Table has a robust design that ensures stability and safety for a broad range of patients. It has a maximum load of 360 kg in normal position, and 180 kg without any restrictions – ideal for most patient populations.



Designed for universal use

A flexible mobile table can serve many purposes. Manual longitudinal shift improves intra-operative diagnostics and treatment. It delivers better access for the surgeon while simultaneously improving accessibility for the C-arm. 360° imaging is possible when paired with carbon fiber accessories.


The Maquet Lyra Mobile OR Table is designed to work for almost all surgical disciplines, increasing the options for each OR. Plus, it's compatible with all existing Getinge Mobile OR Table accessories that you may already have, maximizing the return on your investment.





Versatile angles, tilts, and heights

A 350 mm height range supports a variety of procedures, whether the surgeon is seated or standing. This improves ergonomics reducing strain and minimizing the risk of workplace injury. A broad range of angles and tilts can be combined to give surgeons the best surgical site access with the least patient trauma. This can improve outcomes and reduce overall recovery time.

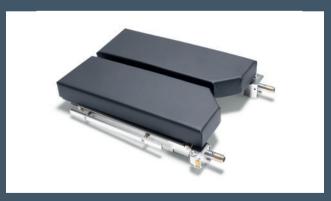

Improved throughput between procedures

Gas-spring assisted leg plates can fold around the column, remaining attached during gynecological procedures. This reduces reconfiguration time and helps to increase throughput.

MAQUET LYRA MOBILE OPERATING TABLES 5

Improving comfort and safety

For staff and patients



Height range from 600 mm to 950 mm


Good ergonomic conditions are important for keeping staff healthy and happy. Proper ergonomics not only reduces the risk of workplace injury, but also minimizes the fatigue that can lead to clinical errors. Maquet Lyra is capable of a height range of 350 mm to meet ergonomic needs.

Excellent clinical access

A concave base and smart design of the column and table top allow the surgical team to get close to the patient – without positional strain. The team can work comfortably and confidently.

Highly maneuverable

The Maquet Lyra Mobile OR Table is mounted on four double swivel castors for easy maneuverability in straight lines or laterally. This makes the table easy to transport and position.

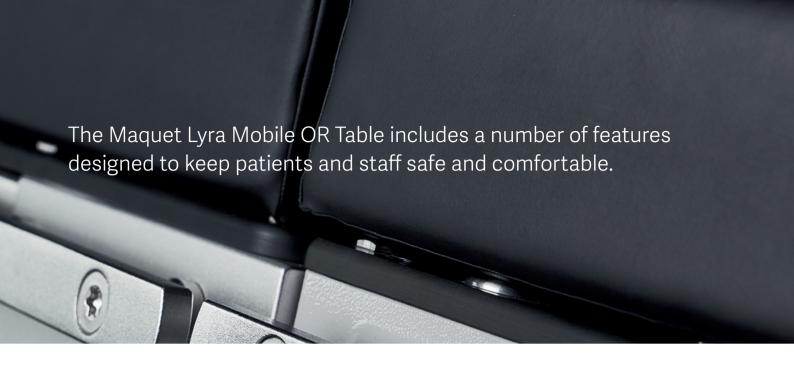



Emergency fail-safe technology

Even in the unlikely event of a table software blackout, the patient remains secure and the table can be safely controlled. The override panel is hard-wired to a separate circuit, with no software in between.

Fewer electrical components

The Maquet Lyra Table has fewer electrical components, simplifying maintenance for reliable and cost-effective long-term use.

Specially designed rechargeable batteries

The rechargeable batteries are long-lasting and reliable. A single charge lasts about one week.

When it comes to safety, there's no room for error. The Maquet Lyra Mobile OR Table has built-in safety features to prevent accident or injury.

Positioning possibilities

A flexible table can improve functionality and increase the cost-effectiveness of your OR



Struma position



Spinal surgery in prone position with optimum access for the C-arm



Femur treatment



Park bench position

Maquet Lyra offers several patient positioning possibilities by offering optimal reverse or normal positioning together with a wide range of angles and lateral tilt combinations. This delivers increased access in various surgical positions. The manual longitudinal shift improves intraoperative diagnostics and treatment. It delivers both increased access for the surgeon, and simultaneously improves accessibility for the C-arm. 360° imaging is possible when paired with carbon fiber accessories.



Neurosurgery and vascular procedures



Gynecological and urological surgery



Shoulder surgery in beach chair position



Kidney procedures

MAQUET LYRA MOBILE OPERATING TABLES

Technical description


and construction features

General construction features

- Stable base construction with four double swivel castors for easy movement and maneuvering (base can be locked via foot pedal)
- Base cover made of SMC (sheet moulding composite)
- The override panel is connected to a seperate circuit in order to ensure that the table can be safely controlled even in the unlikely event of a table software blackout
- · Identical interfaces on normal and on reverse side

Electrical data

- Specially designed rechargeable batteries, with a single charge lasting about one week in the operating room
- · Electronic charge monitoring with visual indicators
- Batteries recharged from the mains supply
- Voltage from 100/100–120/127/200/220/230–240 V, 50/60Hz, over mains cable
- Safety class II, type B; the enclosure leakage current meets the requirements of the patient leakage current for type B conditions as per EN 60601-1

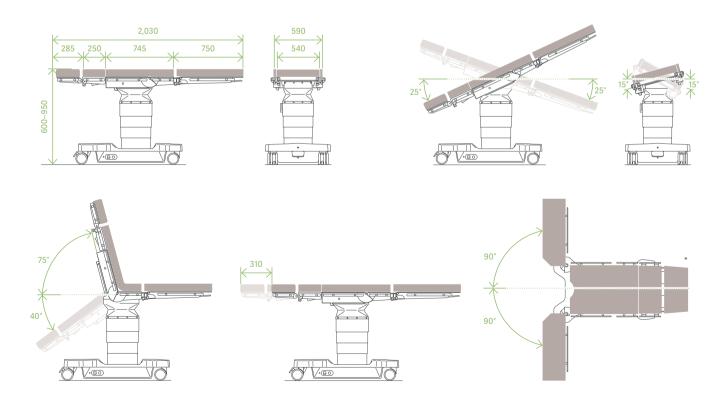

All dimensions are in millimeters

^{*}Table adjustment possible when paired with leg plate 1118.60X0

Technical data

	Maquet Lyra 1118.10XX
Length of operating table top with standard back plate, head rest, and leg plate	2030 mm/80"
Length of operating table top without accessories	745 mm/29"
Width without side rails	540 mm/21"
Width across side rails	590 mm/23"
Net weight (without accessories)	190 kg/419 lbs
Maximum overall load (patient and accessories)	360 kg/794 lbs with restrictions 180 kg/397 lbs without restrictions

Adjustment range

	Maquet Lyra 1118.10XX
Height without padding	600–950 mm / 24–37"
Trendelenburg/Reverse Trendelenburg	25°/25°
Lateral tilt	15°
Lower back plate	+75°/-40°
Leg plate (manual)	-90° (down)*
Manual longitudinal shift	310 mm**/12"

Base brake mechanism (lock/unlock)

11 MAQUET LYRA MOBILE OPERATING TABLES

 $^{^{\}star}$ Table adjustment possible when paired with leg plate 1118.60X0 ** Longitudinal shift is only applicable for Maquet Lyra 1118.10K0/F1



With a firm belief that every person and community should have access to the best possible care, Getinge provides hospitals and life science institutions with products and solutions aiming to improve clinical results and optimize workflows. The offering includes products and solutions for intensive care, cardiovascular procedures, operating rooms, sterile reprocessing and life science. Getinge employs over 10,000 people worldwide and the products are sold in more than 135 countries.

This document is intended to provide information to an international audience outside of the US.

 $\textbf{Maquet (Suzhou) Co., Ltd.} \cdot \text{No.158 Fangzhou} \\ \underline{\text{Road } \cdot \text{Suzhou Industrial Park} \cdot 215024 \, \text{Suzhou} \cdot \text{China} \cdot + 86 \, (0)} \\ \underline{\text{512 62839880}} \\ \underline{\text{100 100 Suzhou}} \cdot \underline{\text{100 Suzhou}} \cdot$